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24.950.10

24.950.12

ZEPF Drop-Control 46.007.02 Blade Holder DISMANTLING, CLEANING AND MAINTENANCE

Dear Customer!

We thank you very much for having chosen this high quality instrument from Helmut Zepf Medizintechnik GmbH. To get the best from your instrument, please follow the instructions below:

DISMANTLING

The ZEPF  Blade Holder has to be dismantled easily **before** cleaning, as shown.

Attention

Don't remove the spring

CLEANING

For all cleaning processes, please consider our application guidance of autoclavable instruments and accessories according to DIN EN ISO 17664, in the currently valid version. See document "0703091007_REPROCESSING_HZ_BA_e", downloadable from our website www.zepf-dental.com.

Pre-Treatment: Use cleaning brush with nylon bristles when incrustations are heavy.

Cleaning: Manual or mechanical –
ATTENTION: Cleaning in a thermal disinfectant:
standing vertically, thread downwards.

Rinsing: Please use deionized water to avoid water spots.

Drying: Use compressed air or air-dry (manual preparation).

MAINTENANCE

Oiling with Medical White Oil, as shown .

ASSEMBLY

Assembly is made in reverse order to the dismantling.

RECOMMENDED CARE PRODUCTS

Cleaning Brushes with nylon bristles

85.903.01 double-ended, sterilizable, packing unit: 3 pieces

85.903.03 for aspirator Ø 1.5 mm

85.903.04 for aspirator Ø 3 mm and Ø 5 mm

 **Medical White Oil** in a 50 ml dispensing bottle and in an Oil Pen, 12 ml

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